



Battery Interrupter

You don't know how to adapt a toy? No problem! We have a solution!

What is it?

Battery interrupters can be used with most toys that have a battery with an on/off switch. The purpose of using a battery interrupter is to allow a person to access and use a device with an adaptive switch. The battery interrupter is placed between the battery and the spring within the battery compartment to be connected to a capability switch. The interrupter will then connect to the batteries after the switch is pressed. This allows the toy to perform its functions as if the toy is turned on.

Instructions:

1. Place Battery Interrupter disc against the battery compartment wall toward the negative end. (Make sure "silver bump" is facing outward within the compartment.)
2. Place the battery in the compartment.
3. The silver bump should fit in the dent of the negative side of the battery.
4. Close the lid of the battery compartment.
5. Plug switch into the jack of the battery interrupter.
6. Now the device should be controlled by the switch.

Ways to use:

- With any toys/devices with AA or AA batteries and an on/off switch
- Bubble machine
- Vibrating pillows
- Sing along stuffed animals



Follow this QR code
for an Amazon link to
Battery Interrupters

Follow this QR code
for an YouTube link on
Battery Interrupters

